



Size:-11.024" x 9.843" inches



HAIKU™

Inflation Device

For the well versed interventionalist.

- ❖ Intuitive and ergonomic design to facilitate stress-free hand-eye coordination.
- ❖ Efficient Intra-cam locking mechanism allows maintaining pressures and guarantees fast automatic release.
- ❖ EasyView™ pressure gauge for strain-free reading.
- ❖ Anti-slip body for grip stability during use.
- ❖ Registers pressures immediately without time delay.
- ❖ Accurate stable pressures with high degree of repeatability.

Ordering Information:

Catalogue Number	Product Description	Max. Inflation Pressure
HKU00030	Haiku™ Inflation Device	30 ATM

Meril

More to Life

Meril Life Sciences Pvt. Ltd.

Survey No. 135/139, Bilakhia House,
Muktanand Marg, Chala,
Vapi 396 191. Gujarat. India.
T +91 260 3052 100
F +91 260 3052 125

Meril GmbH.

Bornheimer Strasse 135-137,
D-53119 Bonn.
Germany.
T +49 228 7100 4000
F +49 228 7100 4001

Meril South America

Doc Med LTDA
Al. dos Tupiniquins,
1079 – Cep: 04077-003 – Moema
Sao Paulo. Brazil.
T +55 11 9738 04868

E askinfo@merillife.com W www.merillife.com

EU Representative

Obelis S. A.
Bd, General Wahis 53,
1030, Brussels, Belgium.
T +32 2 732 5954
F +32 2 732 6003
E mail@obelis.net



Haiku™ and EasyView™ are registered trademark of Meril Life Sciences Pvt. Ltd.

HKU/BROCHURE/002/MLS/20120320/IND



HAIKU™

Inflation Device

Meril
More to Life

HAIKU™

Inflation Device





Size:-11.024" x 9.843" inches



HAIKU™ Inflation Device

For the well versed interventionalist.



EasyView™ Pressure Gauge Dial-Max Pressure 30 ATM

20 ml Syringe


Cam
To Unlock - Push Down
To Lock - Release

Male Rotating Adaptor

Anti-Slip Grip


Ergonomic Piston Handle
Inflation - Clockwise

4 simple steps to use Haiku™ Inflation Device

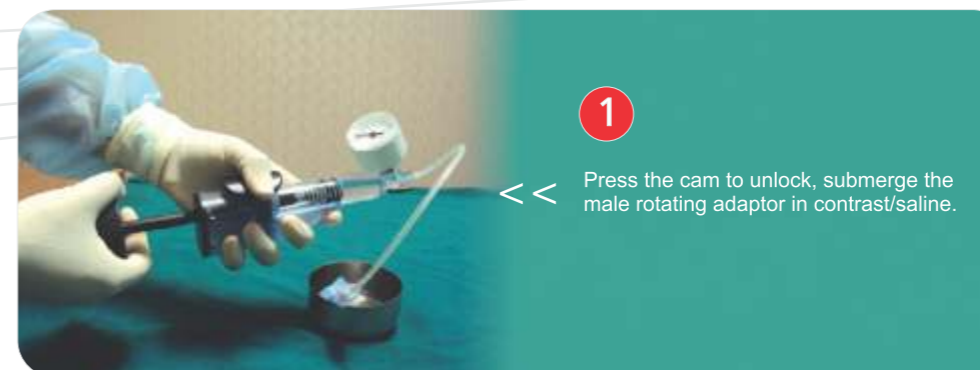

1

<< Press the cam to unlock, submerge the male rotating adaptor in contrast/saline.

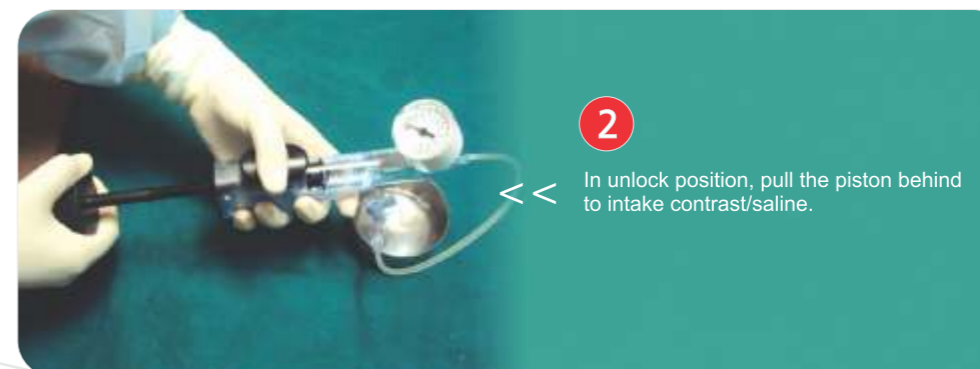

2

<< In unlock position, pull the piston behind to intake contrast/saline.



3

<< After removing air bubbles, release the cam to lock. Inflate by rotating the piston clockwise.



4

<< For rapid deflation, press the cam lock and pull piston handle simultaneously behind.



Each box contains:
One Haiku™ Inflation Device



Only the contents of the inner
shapeless tyvec pouch are sterile

